PL

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USER MANUAL BREATHALYZER HT8G010

PRODUCT INTRODUCTION:

HT8G010 is a portable breathalyzer. The core components of the instrument use semiconductor sensors, which have the advantages of high sensitivity, high stability and easy calibration. The instrument adopts LCD display. This product is suitable for detecting drunk driving or alcohol detection before work, so as to effectively avoid accidents and protect people's life and property safety.

PROFESSIONAL BREATHALYZER

Before use, please read the product instructions carefully and strictly follow the instructions.

Test results vary from person to person that people's reactions to alcohol are inconsistent. Test results are for reference only and should not be used as a criterion for judging whether or not to drink and drive.

Manufacturers and sellers will not bear any responsibility for illegal acts.

REFERENCE

BAC: Blood Alcohol concentration

BAC: Blood Alcohol concentration

Drinking Driving: refers to the driving behavior of vehicle drivers whose blood alcohol content is greater than or equal to 20mg/100m and less than 80mg/100m.

Drunk Driving: refers to the driving behavior of vehicle drivers whose blood alcohol content is greater than or equal to 80mg/100m.

INSTRUCTIONS OF ALCOHOL DETECTOR

Power on

1. Push the air port upwards, the backlight and the display screen will light up at the same time, all the contents will be displayed and the buzzer „beeps” to prompt the power on. After power is on, the blue indicator will be always on.
2. LCD will displays „Warm up” and a 10-second countdown. After the countdown ends, the buzzer will „di” to prompt to enter the next stage at the same time.

INDICATOR DIAGRAM:

If the breathalyzer has not been used for a long time, the sensor surface will be oxidized.

So when the breathalyzer is taken out of the package for the first time or the breathalyzer has not been used for a long time, the user needs to preheat the product as many times as possible, and then perform the alcohol test.

TEST:

1. The display will show „Blow” and a 10-second countdown. After the countdown is over, the buzzer „di” will prompt you to enter the result display.
2. During the 10-second countdown, when air containing alcohol is detected, the countdown will jump directly to the 4-second blow countdown detection and update the countdown.
3. Test results are displayed in units of %BAC and g/L.
4. When the test result is less than 0.02%BAC (or 0.2g/l), the green indicator light will be always on, and the buzzer will not sound, indicating that there is no abnormality
5. When the test result is within the following range, the yellow indicator light will be always on, the „Caution” icon on the display will flash, and the buzzer will have a slow sound.
0.02%BAC(or0.2g/l)
<0.08%BAC(or0.8g/l)
6. When the detection result is greater than or equal to this range, the red indicator light will be always on. The “Danger” icon on the display screen will flash, and the buzzer will have a rapid sound.
≥0.08%BAC(or0.8g/l)

INDICATOR DIAGRAM:

NOTES

1. Don't take the test until at least 20 minutes after drinking alcohol, as it takes about 20 minutes for alcohol to be absorbed from the digestive system into the bloodstream. When tested immediately after drinking alcohol, the results only reflect the current level of alcohol in the mouth, not in the blood.
2. In order to make the test result accurate, please wait for at least 20 minutes before taking the test after drinking. The test time should be more than 3 minutes from the last test time.
3. Do not directly wipe the product with chemicals or corrosive solution, to avoid liquid flowing into the blowpipe.
4. The machine will not work normally if there is a high concentration of interference gas.
5. If the test is performed under low power, the test value will have a certain deviation from the actual value.
6. It is strictly forbidden to place and use it in a closed environment with corrosive gases such as pigments, pesticides, alcohol, etc or air pollution.
7. Unauthorized repair and damage to internal parts will lead to product failure.
8. Prevent the machine from falling down from a high place or being subjected to violent vibration.

SHUTDOWN

In the power-on state, press the button for 1 second to shut down the device, or wait for the end of the test results then the device will automatically enter the shutdown state and shut down. The display screen „OFF” flashes twice, and the buzzer drops twice to prompt the shutdown.

LOW BATTERY

If low battery is detected when power on, the device will not be able to test normally. The „battery box” icon on the display will flash 5 times, the buzzer will beep quickly for 5 times, and then enter the shutdown state.

ALCOHOL CONCENTRATION CHART

Number	(BAC) Blood alcohol concentration				(BrAC) Breath Alcohol concentration	
	Mg/100ml	%	‰	g/l	Mg/l	Common ppm
1	10	0,01	0,1	0,1	0,05	25
2	20	0,02	0,2	0,2	0,10	50
3	30	0,03	0,3	0,3	0,15	75
4	40	0,04	0,4	0,4	0,20	100
5	50	0,05	0,5	0,5	0,25	125
6	60	0,06	0,6	0,6	0,30	150
7	70	0,07	0,7	0,7	0,35	175
8	80	0,08	0,8	0,8	0,40	200
9	90	0,09	0,9	0,9	0,45	225
10	100	0,10	1,0	1,0	0,50	250
11	110	0,11	1,1	1,1	0,55	275
12	120	0,12	1,2	1,2	0,60	300
13	130	0,13	1,3	1,3	0,65	325
14	140	0,14	1,4	1,4	0,70	350
15	150	0,15	1,5	1,5	0,75	375
16	160	0,16	1,6	1,6	0,80	400
17	170	0,17	1,7	1,7	0,85	425
18	180	0,18	1,8	1,8	0,90	450

19	190	0,19	1,9	1,9	0,95	475
20	200	0,20	2,0	2,0	1,00	500

Common units in various countries:

mg/ 100ml—CN,

%BAC—USA,

g/l—EU,

mg/BrAC—JP

0.08%BAC refers to 0.08g(80mg) of alcohol per 100ml of blood.

MAIN TECHNICAL PARAMETERS:

PROJECT	INDICATORS AND PARAMETERS
Type of Alcohol Sensor:	semiconductor type
Response Time:	≤20S
Measurement Range:	0-0.19%BAC 0-0.95mg/L
Indication Error:	±0.01%BAC
Calibrating Period:	6 months
Working Temperature Range:	-5°C ~ 45°C
Storage Temperature Range:	-20°C ~ 50°C
Blowing Continuity Test:	5 seconds (airflow: ≥10L/min)
Power Supply:	AA battery
Working Current:	150mA
Display:	LCD (liquid crystal display)
Alarm Mode:	buzzer
Machine Size:	105mm (length)*32mm (width)*16 mm (thickness)
Instrument Weight:	about 30g

DECLARATION

1. Regardless of whether the product is used or not, neither the manufacturer nor the seller will bear any responsibility for illegal acts. For example: drunk driving, working at height.
2. Everyone reacts differently to alcohol, so the test results are for reference only and cannot be used as a standard to determine the results

Product Name: Portable Alcohol Tester

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